



PRIMO HeelShield II Offloading Heel Protector

The PRIMO HeelShield II Offloading Heel Protector is a systematic treatment to offload or “float” the heels of non-ambulating patients in order to reduce the risk of heel pressure ulcers.

CAUTION: DO NOT WALK OR STAND IN PRIMO HEELSHIELD II!

APPLICATION INSTRUCTIONS

- 1 Record Patient’s name on NAME label. Release all hook & loop closures. Place heel in hole (NOTE: it may be helpful to bend the patient’s knee slightly).



- 2 If also using a compression device, route tubing through port hole, ensuring that tubing neither kinks nor is in contact with patient’s skin.



- 3 Attach the two shin straps to the opposite side of the boot. Tighten black straps for a snug fit, but do not over-tighten such that blood flow is restricted.



- 4 Place foot in neutral or slight dorsiflexion and secure foot positioning straps.



- 5 Feel underneath to make sure the patient’s heel is floated. If not, undo straps and re-apply.



SIZING CHECK:

- ✓ If heel touches bed, use bariatric size
- ✓ If straps touch shin, use bariatric size
- ✓ If snug fit not possible, use small size

- Use only on non-ambulating patients. Do not allow patient to walk or stand while wearing PRIMO HeelShield II
- Remove PRIMO HeelShield II periodically to inspect patient’s skin
- Report any pain, swelling, changes in sensation, or unusual reactions to a clinician
- Do not use without the order of a licensed practitioner

Cleaning: PRIMO HeelShield II can be wiped down with most disinfectant wipes. It can also be cleaned by washing instructions below.

Washing Instructions: PRIMO HeelShield II can be washed in any washer using standard detergent. Do not exceed temperature of 70 degrees centigrade or 160 degrees Fahrenheit. DO NOT BLEACH. PRIMO HeelShield II can be air-dried or dried in dryer subject to temperature restrictions above.

CAUTION: Federal law (USA) restricts this device to sale by or on the order of a practitioner licensed by applicable law to use or order the use of this device.

WARNING: This product should be used under strict supervision of a medical professional. Incidents of pain, swelling, changes in sensation, or unusual reactions should be reported to a clinician.

WARRANTY: PRIMO’s sole warranty guarantees the workmanship of the supplied product. PRIMO will replace any defective product upon customer receipt of the product. PRIMO makes no other warranty of any kind. The manufacturer shall not be liable for any loss, damage, expense, or injury arising from the use of this product.