

# Glide & Guide Patient Turn System INSTRUCTIONS

- 1 Begin by lowering the bed rails to prepare the Glide & Guide Turn System for use. Adjust the bed to comfort level and lock the bed brakes for stability.



- 2 FOR GLIDE & GUIDE TOP STRAP & SIDE STRAP SHEET ONLY: Adjust the versatile straps at either the head or side of the bed; depending on the bed type.



- 3 Align the Glide & Guide Turn Sheet with the patients shoulders. Center the Glide & Guide Lift Sheet under the patient by rolling them away from you or placing the patient onto one side. Follow your patient handling policy.



- 4 Use the Glide & Guide Turn Sheet handles to gently reposition and adjust patient on the bed. Angle the patients' body 30 degrees in preparation for the Body Positioning Aligner(s).



- 5 Place the short wedge against the upper torso of the patients body and place the long wedge along the lower portion of patients body. Be sure and offload the sacrum making it free from contact.



- 6 Smooth out all imperfections and add the Super Absorbent Under Body Pad as needed. Adjust the bed accordingly to patient positioning policy; and raise the bed rails.



## CAUTION

- Rotate Patient from side to side q2h.
- ALWAYS follow your facilities' patient handling policy/procedure while utilizing the Glide & Guide Turn System.
- Carefully INSPECT the Glide & Guide Turn System before use. Be sure to DISPOSE of and/or REPLACE all unsanitary products and components.